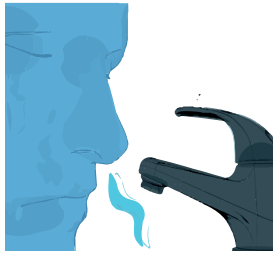
SINUSET®

FILTRO NASALE SINUSET

Please read this instruction leaflet carefully
[Leggere attentamente questo foglio di istruzioni](#)
Bitte lesen Sie diese Gebrauchsanweisung aufmerksam durch

INSTRUCTIONS FOR USE - ISTRUZIONI PER L'USO - GEBRAUCHSANLEITUNG

1


Eng. Wash the nostrils with clear water and dry them carefully out;

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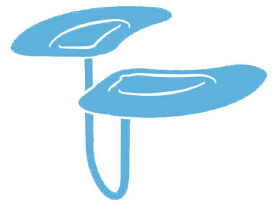2

Eng. Assure that the longer part of the filters look downwards when inserting the nose-filter in the nostrils;

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3

Eng. Push the nose-filter gently in order to make it fit perfectly in the nostrils;

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4

Eng. Each nose-filter can be used from 2 - 5 days depending of the air pollution where it is exposed. After use place it back into the separate place in the box for reuse in order to avoid dust accumulation or the contamination with the others clean nasal-filters.

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Eng. Keep the nose-filter in a cool place away from sunlight and heat. After use place it back into the separate place in the box for reuse in order to avoid dust accumulation or the contamination with the others clean nasal-filters.

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Eng. The nose-filter should not be worn on open wounds in order to avoid undesirable effects. Before using the nose-filter, cure the open wounds. Consult your doctor for proper treatment of the open sores in the nostrils.

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Eng. Dispose of the used nose-filter hygienically. The nose-filter is 100% Biodegradable. Throw the waste in the bin for plastic. Do not flush it down the toilet.

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Manufactured in Italy **Prodotto in Italia** Hergestellt in Italien

LEM SRL

Streda de Pareda, 54 - 38032 Canazei (TN)
C.F. e P.IVA: 02460400225 Email: info@sinuset.it
www.sinuset.com